



Assembly instructions for Support Stool

Parts list for Support Stool

1. Metal frame with seat – 1 set.

Before assembly please make sure if the box contains all necessary parts.



Assembly steps:

- 1. Put the frame on the flat surface.
- 2. In the upper part, unscrew the knob and pull out the seat clamping bar.

ATTENTION!

The seat is assembled backwards for transport purposes

- 3. Turn over the seat part and assemble the bar in the frame.
- 4. Set the seat at desired height and screw the knob.
- 5. Spread the lower part of the frame to its maximum position.